




operating

OPERATING MANUAL

ASSEMBLY/USAGE SIGNATURE SERIES RAISED TOILET SEATS
FOR MODELS:

1640 5" Seat / 1645 5" Seat with Arms

WARNINGS

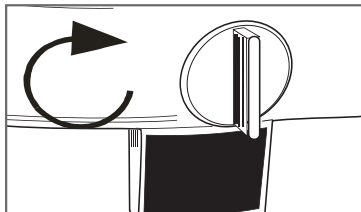

Never overtighten the clamps as this could cause damage to the toilet bowl or the seat itself.



Check your raised toilet seat every week to ensure all clamps are securely tightened.



Only use the raised toilet seat on toilets that meet the specifications. Ensure the seat rests well on the toilet frame.




DO NOT exceed weight capacity!




DO NOT USE THE RAISED TOILET SEAT IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!

CARE AND USE

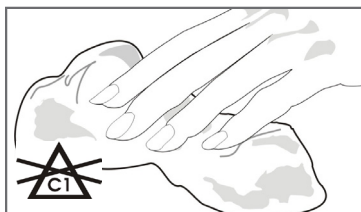


DO NOT use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only.

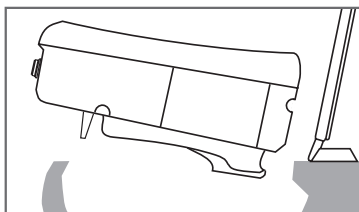

This raised toilet seat will only fit standard toilet seats (approximate size: 12 $\frac{3}{4}$ " x 10 $\frac{1}{4}$ "/32cm x 26cm).



Supervision or assistance when using seats is recommended for users with limited mobility.

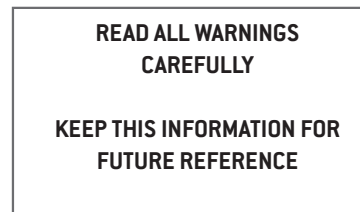


ASSEMBLY




Before installing your raised toilet seat, make sure your toilet is dry and dirt free.




No tools are required.

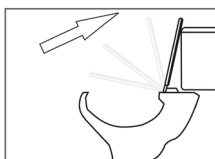


Lift lid and seat of toilet. Some toilet bowls may require that the lid and seat assembly be removed for proper installation.

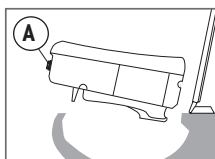

Place the raised toilet seat onto the rim of the bowl with the knob (A) facing forwards.

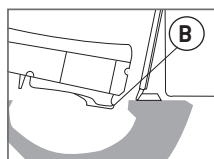

Make sure that the rear flanges (B) are under the rear edge of the toilet bowl trim.

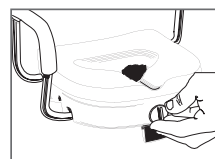

Tighten the raised toilet seat by rotating the adjustment knob by hand clockwise until the seat is secure. **DO NOT OVERTIGHTEN.**

NOTE:
To remove your raised toilet seat turn knob counterclockwise. Lift the front of the raised toilet seat until it is released from the bowl rim, then pull forward and remove.

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE
Your Life. Your Way.