



# operating

## OPERATING MANUAL

ASSEMBLY/USAGE LOOK SERIES RAISED TOILET SEATS  
FOR MODELS:


1662 2" Seat

1663 2" Seat with Lid


1664 4" Seat

1665 4" Seat with Lid

### WARNINGS

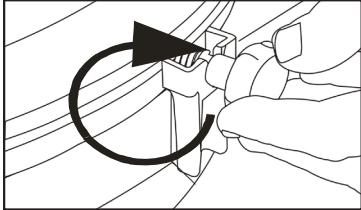

**Never** overtighten the clamps as this could cause damage to the toilet bowl or the seat itself.

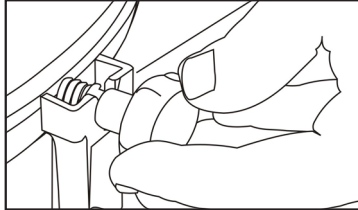

**Check** your raised toilet seat every week to ensure all clamps are securely tightened.



**Only** use the raised toilet seat on toilets that meet the specifications. Ensure the seat rests well on the toilet frame.




**DO NOT** exceed weight capacity!

**DO NOT USE THE RAISED TOILET SEAT IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!**

### CARE AND USE




**DO NOT** use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only.

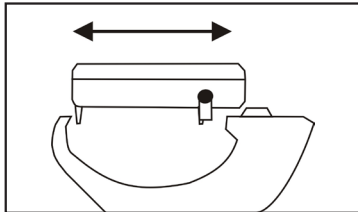

**This** raised toilet seat will only fit standard toilet seats (approximate size: 12<sup>3</sup>/<sub>4</sub>" x 10<sup>1</sup>/<sub>4</sub>"/32cm x 26cm).




**Supervision** or assistance when using seats is recommended for users with limited mobility.

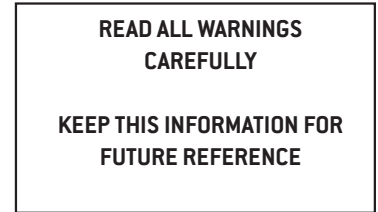


### ASSEMBLY




Before installing your raised toilet seat, make sure your toilet is dry and dirt free.

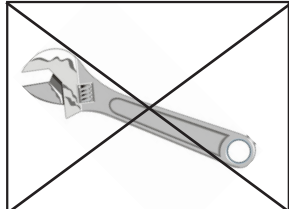


No tools are required.

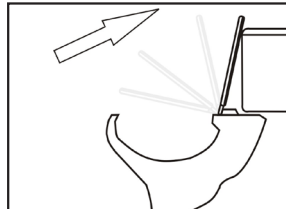

Lift the existing seat and place the raised toilet seat onto the toilet bowl.

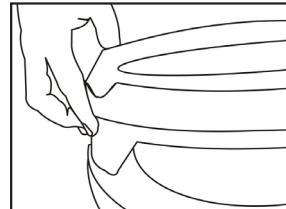

Position the front lip against the front of the toilet bowl.

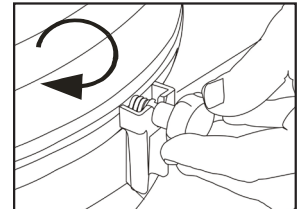

Turn both adjustment knobs clockwise until the rubber buffers touch the toilet bowl.

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THIS PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE CANADA, INC.  
10-155 Colonnade Road  
Ottawa, ON K2E 7K1

MADE IN CHINA

Human Care Canada Inc. All rights reserved. This manual may not be reproduced in part or in whole without the written permission of the publisher. [REV: JULY 2013]



**HUMAN CARE**  
Your Life. Your Way.