




operating


OPERATING MANUAL

ASSEMBLY/USAGE SHOWER SEATS
Models #1460 / #1465 (with back)

COMPONENTS

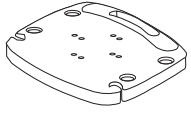

1 x Seat

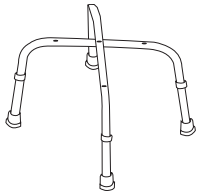

2 x Legs
(Marked ① & ②)



1 x Backrest
(Model #1465 Only)



4 x 42mm Screw
4 x Washers

1 x 50mm Screw
1 x Washer and Lock Washer




DO NOT USE ANY PIECE IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS


READ ALL WARNINGS CAREFULLY

KEEP THIS INFORMATION FOR FUTURE REFERENCE


WARNINGS

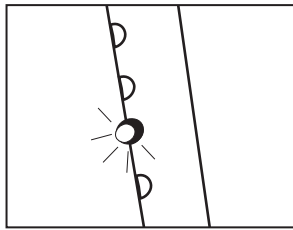


Ensure all push buttons are securely in position before sitting.

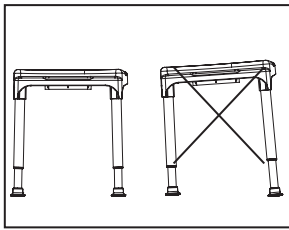

Ensure legs are adjusted to be level before use to avoid possible injury.

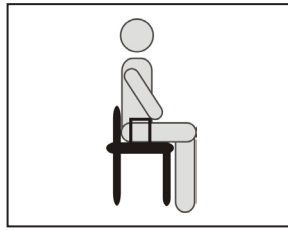

DO NOT stand on seat. Legs must firmly touch the ground when seated.

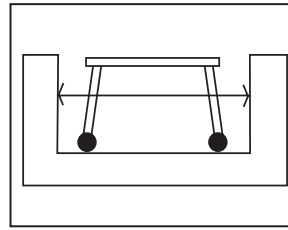


Only use on tub/shower floors that are wide enough to ensure stability of all four legs.




DO NOT exceed weight capacity!


CARE AND USE

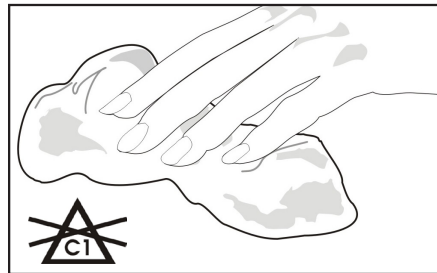


DO NOT use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only. Rinse well after use.

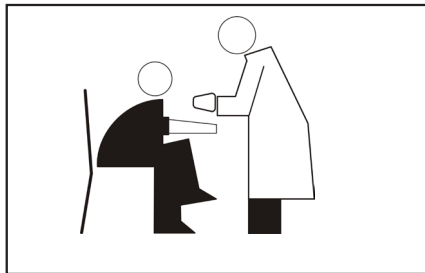


Supervision or assistance when using seats is recommended for users with limited mobility.

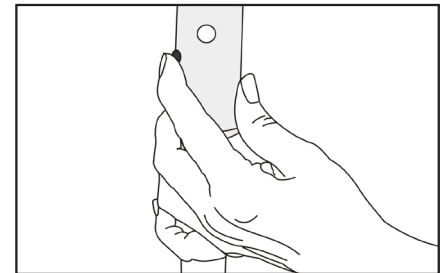


Periodically check the legs to ensure that all push buttons protrude fully through the adjustment holes in the legs.

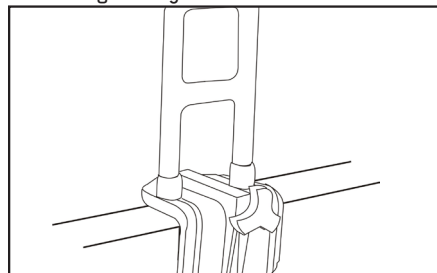

Increase safety by using a slip resistant bath mat under the seat. Shower seats are not to be used for assistance in entering/exiting the bathtub. Use a bathtub safety rail to assist you in sitting down if necessary.

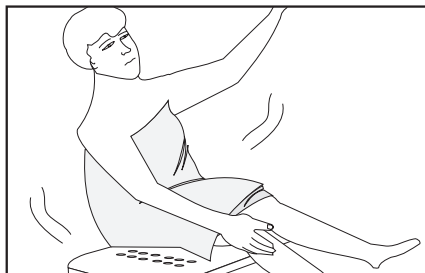

Ensure your weight is evenly distributed when sitting. Improper or uneven loading may result in the chair tipping over and/or falling.

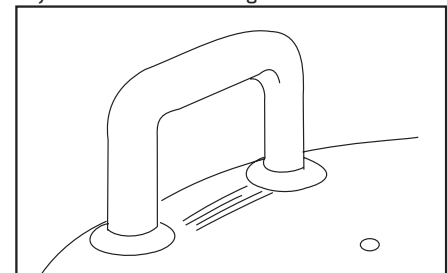

Use only authorized parts/ accessories with your chair. **SOLD SEPARATELY**



HUMAN CARE
Your Life. Your Way.




OPERATING MANUAL


operating

ASSEMBLY/USAGE SHOWER SEATS
Models #1460 / #1465 (with back)


ASSEMBLY

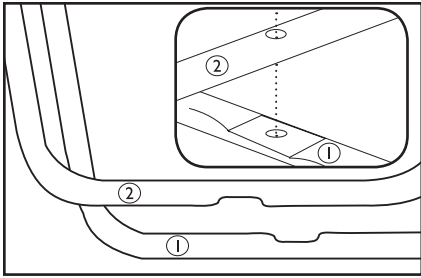

Legs are marked with a number ① and number ②. Seat is also marked to indicate where leg is to be placed.

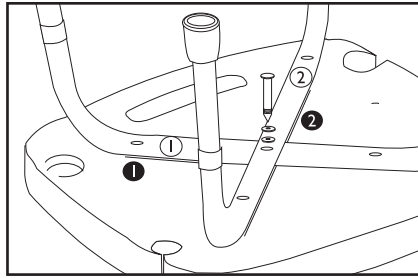


Flip over the seat and place leg marked ① on the seat indent marked ①. Place leg marked ② on the seat indent marked ②. Use the 50mm screw and washer set and screw into the center hole.

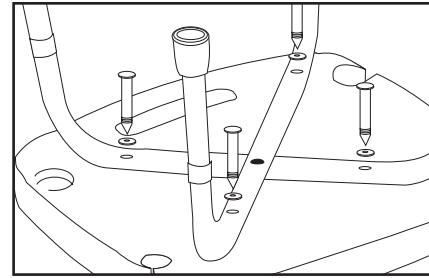


Screw in the 42mm screws into the remaining four holes. **Caution: Do not overtighten as this will strip the holes in the seat. If the longer screw is used in any of these four holes it will protrude through the seat! This will void the warranty!**




DO NOT USE A DRILL - HAND TOOLS ONLY!

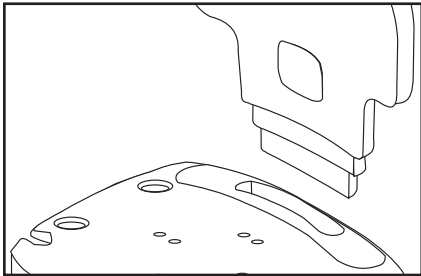

Turn over seat and insert backrest firmly in place by pushing into slot located in the seat.

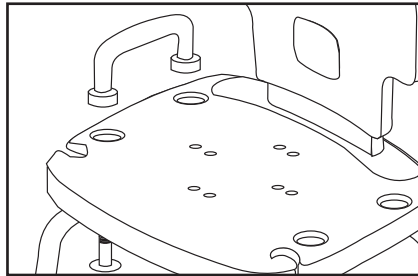

A red handle can be added by resting handle into slots located on seat and holding in place by screwing in from underneath. Red handle can be added to either or both sides.

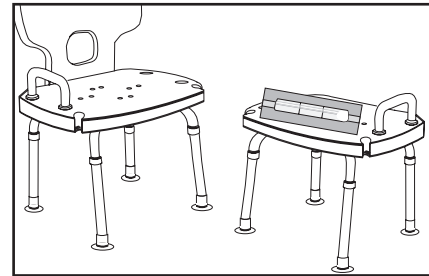

ENSURE SEAT IS LEVEL BEFORE USING!

RED HANDLE SOLD SEPARATELY

CE **KG!**
HUMAN CARE CANADA INC.
10-155 Colonnade Road
Ottawa, ON K2E 7K1
MADE IN TAIWAN

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE
Your Life. Your Way.