




operating

OPERATING MANUAL

ASSEMBLY/USAGE TRANSFER BOARD

Models #1495

COMPONENTS

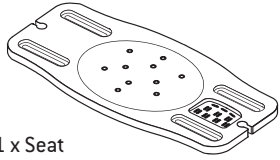

1 x Seat



4 x Footrest



4 x Screws



8 x Rubber Pads




DO NOT USE ANY PIECE IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS


READ ALL WARNINGS CAREFULLY

KEEP THIS INFORMATION FOR FUTURE REFERENCE


WARNINGS

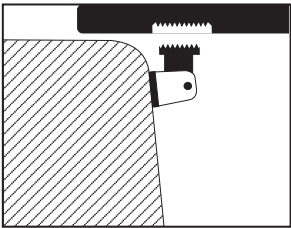


Always fit in the teeth before tightening while assembling footrests.

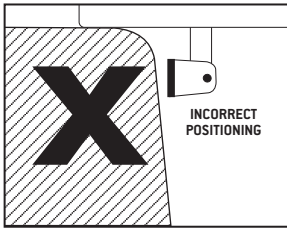

Ensure footrest lays flat against bathtub surface before use to avoid possible injury.

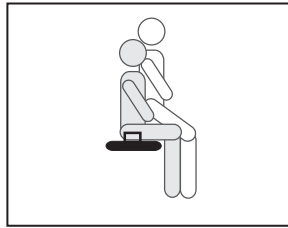

DO NOT stand on board or drop onto board. Your legs must firmly touch the ground when seated.

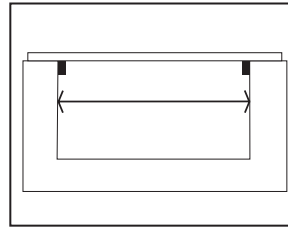


Only use on bathtub that is wide enough to ensure stability of all four footrests.




DO NOT exceed weight capacity!


CARE AND USE

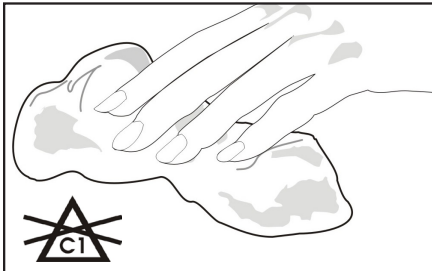


DO NOT use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only. Rinse well after use.

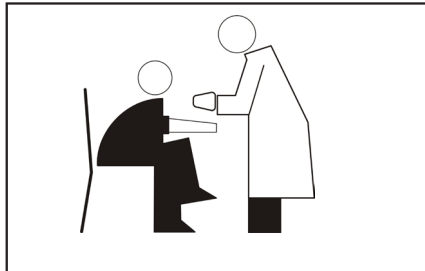


Supervision or assistance when using boards is recommended for users with limited mobility.

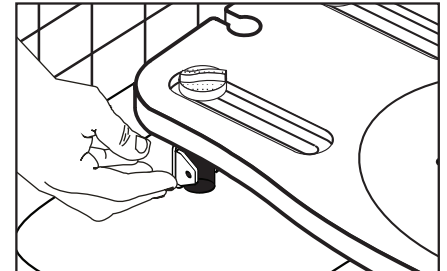


Periodically check the anti-slip pads and footrests to ensure that they are firmly in place.

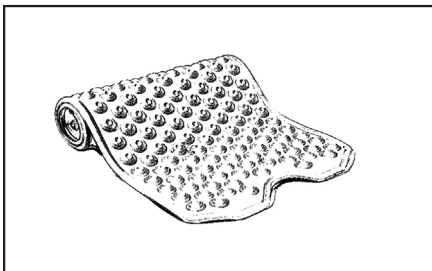

Increase safety by using a slip resistant bath mat in the bathtub.

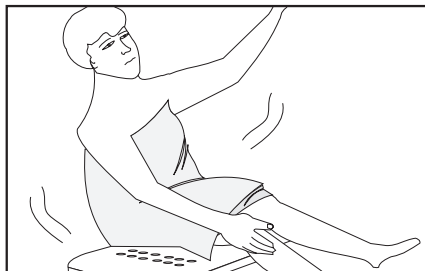

Ensure your weight is evenly distributed when sitting. Improper or uneven loading may result in the board tipping over and/or falling.

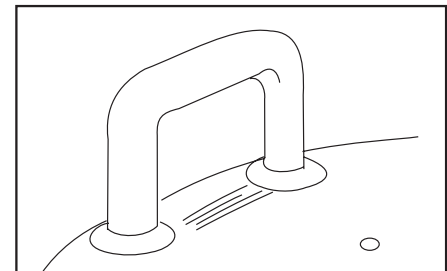

Use only authorized parts/ accessories with your board. **SOLD SEPARATELY**



HUMAN CARE
Your Life. Your Way.



operating

OPERATING MANUAL

ASSEMBLY/USAGE TRANSFER BOARD

Models #1495

ASSEMBLY




ENSURE BOARD IS LEVEL AND SECURELY IN PLACE BEFORE USING!




PAY EXTREME CAUTION ON WET SURFACES!




DO NOT USE IF THERE ARE ANY CRACKED OR DAMAGED COMPONENTS!

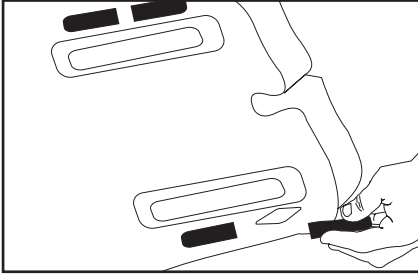


Place anti-slip pad into the holder located on the bottom of the transfer board and press down until firmly in place.

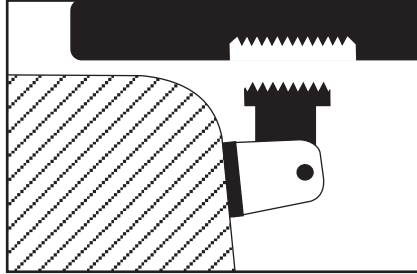


Fit the teeth in the footrest into the teeth on the bottom of the board.

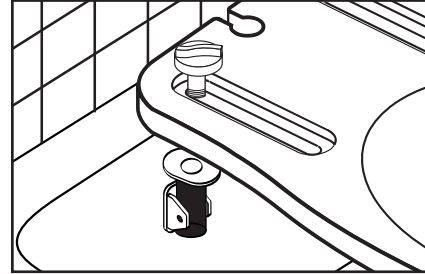


Hold footrests by screwing in place from the top of the transfer board with the plastic screw.

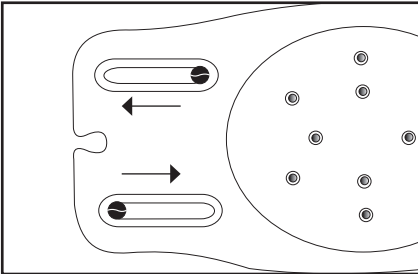

Adjust footrests to bathtub size by sliding to required distance. Once correct position determined, tighten screws by hand.

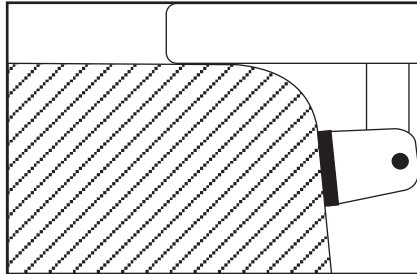

Fit footrest with bathtub surface at a slope. Ensure that the footrests fit snugly against the bathtub wall.

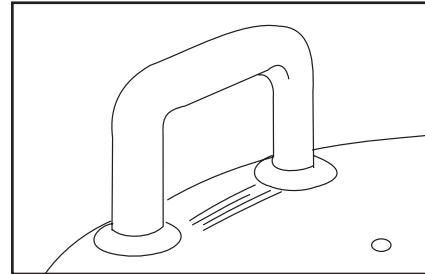

A red handle can be added by resting handle into slots located on board and holding in place by screwing in from underneath. Red handle can be added to either or both sides.



DO NOT OVERTIGHTEN! This will strip the screws and holes in the footrest.

RED HANDLE SOLD SEPARATELY

CE **KG!**
HUMAN CARE CANADA INC.
10-155 Colonnade Road
Ottawa, ON K2E 7K1
MADE IN TAIWAN

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE
Your Life. Your Way.