




# operating


## OPERATING MANUAL ASSEMBLY/USAGE GRAB BARS

FOR: 1000 Series Fixed Flange  
1800 Series Adjustable Flange


### WARNINGS

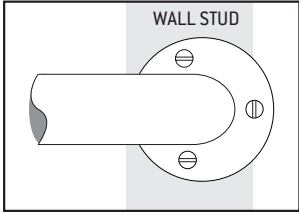

**Ensure** that the grab bar has been installed onto wall studs securely for solid support.

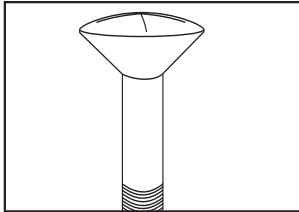

**Use** only the stainless steel screws provided to ensure proper installation and prevent rusting.

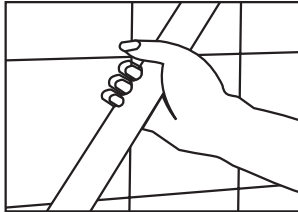

**Test** the grab bar frequently to ensure that it is securely fastened. Check all screws to ensure they are secure.



**DO NOT** exceed weight capacity!




**DO NOT** use chlorine bleach or abrasive cleansers. Use soap and water or a mild detergent.


**! DO NOT USE THE GRAB BAR IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS! NEVER ATTEMPT TO INSTALL THE GRAB BAR YOURSELF!**

### INSTALLATION - FOLLOWING INFORMATION ONLY FOR AUTHORIZED DEALER USE


**READ ALL WARNINGS CAREFULLY**  
**KEEP THIS INFORMATION FOR FUTURE REFERENCE**

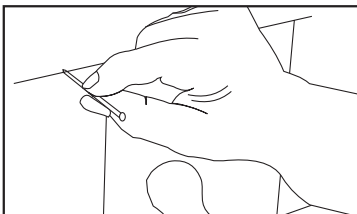

**Locate** two studs on the wall using a stud finder (if available). Studs are spaced 16"/40.6cm apart. Mark location. [If surface is tiled mark at top of tiled surface and proceed to next step].

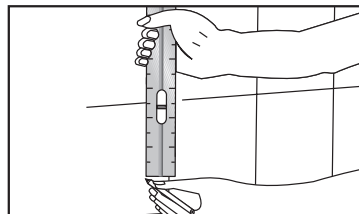


**On tiled surfaces**, use a level and extend the stud marks down to the grab bar location. Place a strip of masking tape to indicate location of studs.

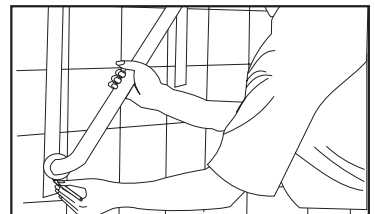

**Mark the location** of the grab bar holes onto the wall (or masking tape). If installing on a tiled surface, use a drill with the appropriate drill bit and drill a hole into the tile until you hit wood.

**! IMPORTANT: INSTALLATION MUST ONLY BE DONE BY A PROFESSIONAL!**

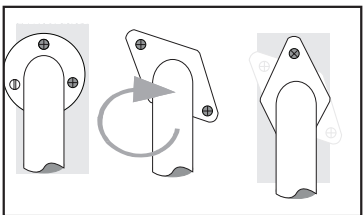

**Two screws must penetrate the studs at least 1"/2.5cm in.** For those grab bars with three screws, use a plastic anchor to secure the third screw. For adjustable flange, rotate flange to align with studs.



**Give** the grab bars a good solid yank to test their holding power.



HUMAN CARE CANADA, INC.  
10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
MADE IN TAIWAN

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



**HUMAN CARE**  
Your Life. Your Way.