




operating


OPERATING MANUAL ASSEMBLY/USAGE GRAB BARS

FOR: 1700 Series Hidden Flange


WARNINGS

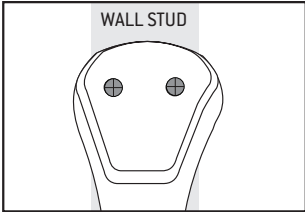

Ensure that the grab bar has been installed onto wall studs securely for solid support.

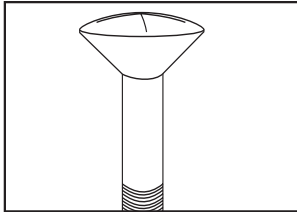


Use only stainless steel screws to ensure proper installation and prevent rusting.

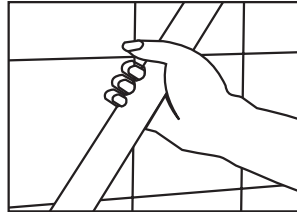

Test the grab bar frequently to ensure that it is securely fastened. Check all screws to ensure they are secure.




DO NOT exceed weight capacity!

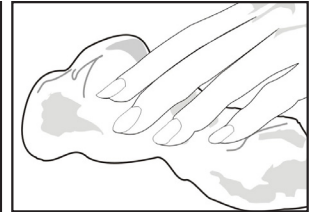


DO NOT use chlorine bleach or abrasive cleansers. Use soap and water or a mild detergent.

! DO NOT USE THE GRAB BAR IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS! NEVER ATTEMPT TO INSTALL THE GRAB BAR YOURSELF!


INSTALLATION - FOLLOWING INFORMATION ONLY FOR AUTHORIZED DEALER USE

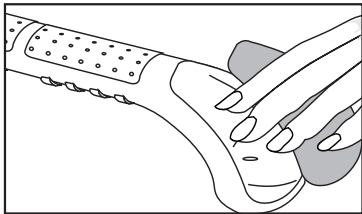


Remove plastic cover from grab bar to expose screw sockets.

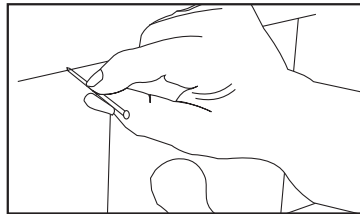


Locate two studs on the wall using a stud finder (if available). Studs are spaced 16"/40.6cm apart. Mark location. [If surface is tiled mark at top of tiled surface and proceed to next step].

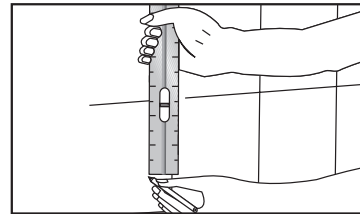


On tiled surfaces, use a level and extend the stud marks down to the grab bar location. Place a strip of masking tape to indicate location of studs.

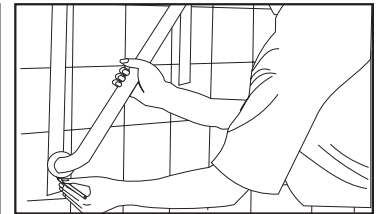

Mark the location of the grab bar holes onto the wall (or masking tape). If installing on a tiled surface, use a drill with the appropriate drill bit and drill a hole into the tile until you hit wood.

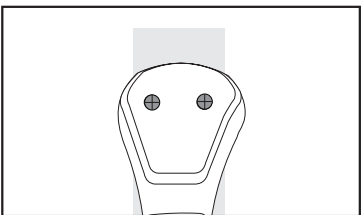

Screws must penetrate the studs at least 1"/2.5cm in.

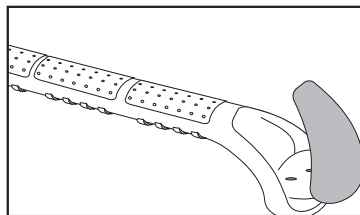

Once installed, cover opening with plastic cap. Ensure it "clicks" into place.



Give the grab bars a good solid yank to test their holding power.

READ ALL WARNINGS CAREFULLY
KEEP THIS INFORMATION FOR FUTURE REFERENCE

! IMPORTANT: INSTALLATION MUST ONLY BE DONE BY A PROFESSIONAL!



HUMAN CARE CANADA, INC.
10-155 Colonnade Road
Ottawa, ON K2E 7K1
MADE IN CHINA

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE
Your Life. Your Way.