




# operating


## OPERATING MANUAL

ASSEMBLY/USAGE SIGNATURE BATH SAFETY RAILS  
FOR MODELS: 1260 Tall / 1265 Regular


### WARNINGS

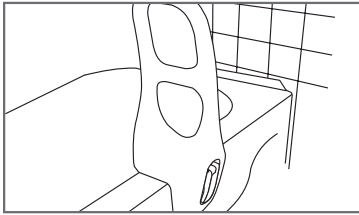

Never overtighten the bath safety rail as this could cause damage to the bathtub or the rail itself.

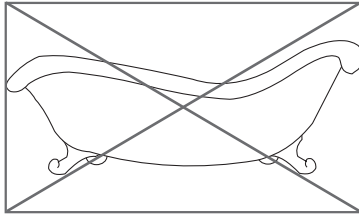

Never install your bath safety rail on a plastic or fiberglass tub. The tub rail will not fit on curved bathtubs.

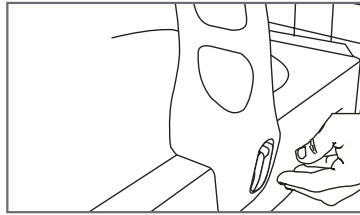


Re-tighten your bath safety rail once a week to maintain proper installation.




DO NOT exceed weight capacity!  
**! DO NOT USE THE BATH SAFETY RAIL IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!**


### CARE AND USE

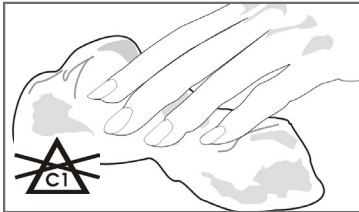

DO NOT use chlorine bleach or abrasive cleansers. Use soap and water or a mild detergent. Rinse well before using it again.

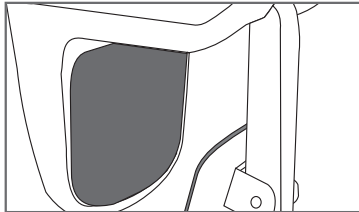


Periodically check the bath safety rail to ensure the rubber pads are securely affixed to the bath safety rail.

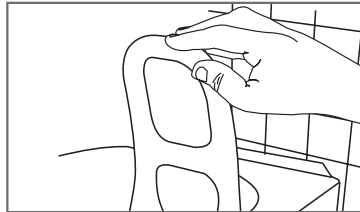


Verify the bath safety rail is firmly in place before each use (should stay in place when pressure applied or when pulled).

**READ ALL WARNINGS CAREFULLY**  
**KEEP THIS INFORMATION FOR FUTURE REFERENCE**


### ASSEMBLY

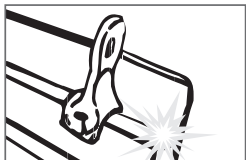


Before installing your bath safety rail, make sure your bathtub is dry and dirt free. The bath safety rail can be mounted at either end of the tub.

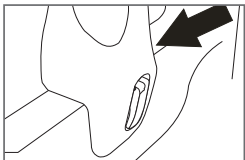


Locate the flattest section of the inner and outer bathtub wall. AVOID curves in the bathtub where bath safety rail cannot properly grip.

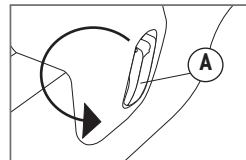

Pull adjustment handle (A) out to use as a lever. Turn handle counterclockwise to loosen until clamp is wide enough to fit bathtub wall.

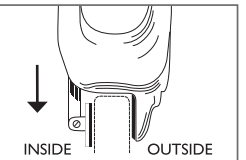

Place the bath safety rail down onto the bathtub wall with the adjustment handle facing the outside of the bathtub.

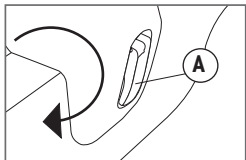

Make sure that the bath safety rail is firmly in position before tightening. Turn adjustment handle (A) clockwise to tighten. With the bath safety rail firmly in place, fold down adjustment handle until it locks into place.



Test mounting by pulling bath safety rail forward, back and side to side. If bath safety rail moves, loosen adjustment handle (A). Check that it is not threaded or ensure that the pads are not loose. If okay, replace at desired position and re-tighten.



**CAUTION:** Verify the bath safety rail is firmly in place before use. If bath safety rail is not securely in place, it may come loose causing injury.

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

613-723-6734



HUMAN CARE CANADA INC.  
10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
MADE IN CANADA



**HUMAN CARE**  
Your Life. Your Way.