





# operating

## OPERATING MANUAL ASSEMBLY/USAGE BATH SAFETY RAILS FOR MODEL: 1240


### WARNINGS

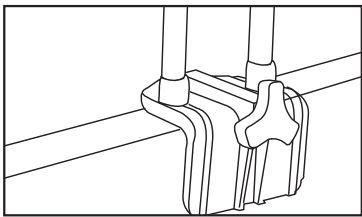

Never overtighten the bath safety rail as this could cause damage to the bathtub or the rail itself.

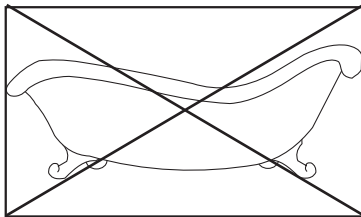

Never install your bath safety rail on a plastic or fiberglass tub. The tub rail will not fit on curved bathtubs.

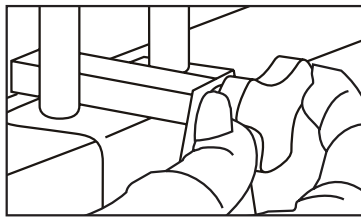

Re-tighten your bath safety rail once a week to maintain proper installation.




DO NOT exceed weight capacity!  
**! DO NOT USE THE BATH SAFETY RAIL IF THERE ARE ANY CRACKS OR DAMAGED COMPONENTS!**


### CARE AND USE



DO NOT use chlorine bleach or abrasive cleansers. Use soap and water or a mild detergent. Rinse well before using it again.

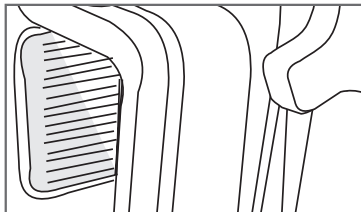


Periodically check the bath safety rail to ensure the rubber pads are securely affixed to the bath safety rail.

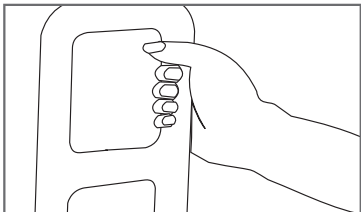


Verify the bath safety rail is firmly in place before each use (should stay in place when pressure applied or when pulled).

**READ ALL WARNINGS CAREFULLY**  
**KEEP THIS INFORMATION FOR FUTURE REFERENCE**


### ASSEMBLY

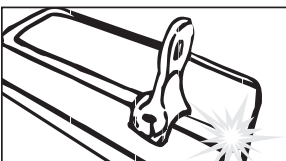


Before installing your bath safety rail, make sure your bathtub is dry and dirt free. The bath safety rail can be mounted at either end of the tub.

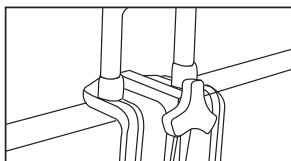

Locate the flattest section of the inner and outer bathtub wall. AVOID curves in the bathtub where bath safety rail cannot properly grip.

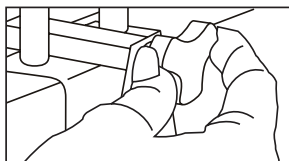

Loosen adjustment knob until bath safety rail is wide enough to fit tub. Place bath safety rail on tub with adjustment knob facing outwards. Turn adjustment knob clockwise to tighten.

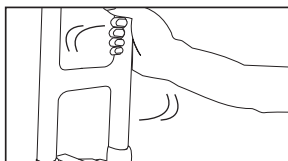

Test mounting by pulling rail forward, back and side to side. If bath safety rail moves, loosen knob, check that the knob is not threaded or the pads loose. If okay, replace at desired position and retighten.



**CAUTION:**  
Verify the bath safety rail is firmly in place before use. If bath safety rail is not securely in place, it may come loose causing injury.

IF YOU HAVE ANY PROBLEMS OR REQUIRE ASSISTANCE WITH THE PRODUCT PLEASE CALL:

**613-723-6734**



**HUMAN CARE**  
Your Life. Your Way.