HUMAN CARE
Your Life. Your Way.

Carl-Oskar & Rebel

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About our products

On delivery

Check the following before using your new product.

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.


The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.

CE-marking

The walking-aid is CE-marked according to the applicable directive for Medical devices. CE-markings are placed on the chassis and contain the following information.

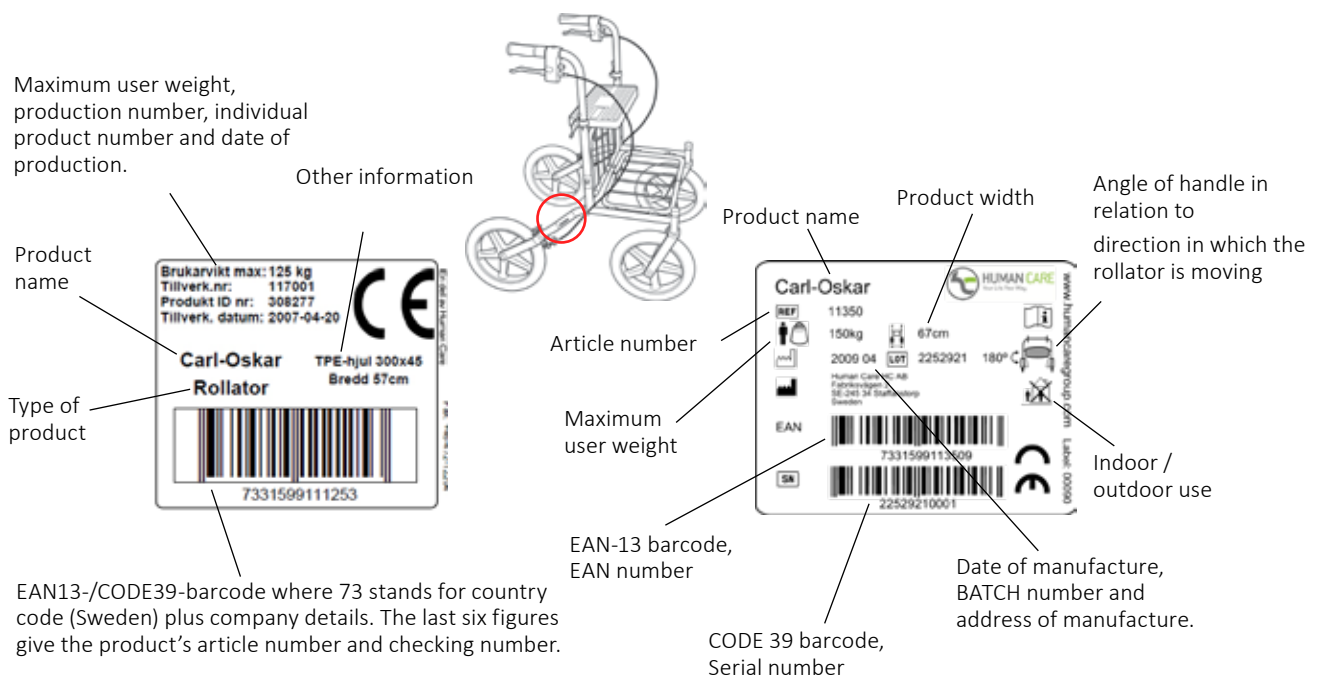The rollator is tested according to ISO 11199-2:2005 (rollator standard) by an independent test institute.

Safety

In the instruction manual, three ways of drawing special attention are used. These ways are highlighted in the following manner.

Attention!

Risk of personal injury or mechanical fault.

Important!

Risk of mechanical damage or fault.

Note!

Important information to ease handling.

Recycling of products

When recycling, the product's chassis is separated from other parts. At separation, the parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials.

The products consist mainly of steel pipe and other metal components.

Examples of parts which consist of combined materials, are the plastic front forks (built up in several layers), wheels and other components, which consist of a composite of materials.

Note!

Different authorities/recycling companies handle recycled materials in different ways. It is thus fitting when in any doubt over the best way to proceed when recycling, to contact the relevant governing body.

Environment friendliness

Human Care aids are manufactured so as to have as little effect on the environment as possible during their lifecycles. This allows for the expired product to be dismantled and for its parts to be recycled. Products with lengthy lifecycles safeguard our sensitive environment and are thus more environmentally friendly.

Human Care HC AB is a part of FTI. FTI – A countywide system for recycling of packaging.

Intended use

Carl-Oskar and Rebel are stable rollators, suitable for use both out- and indoors. With their flexible chassis, they work well even on uneven surfaces. Both rollators have a unique way of folding, enabling them to be stored upright whilst folded. They can be folded and unfolded in one simple operation. Carl-Oskar and Rebel are available as reinforced models for users weighing up to 275 kg.

Technical Data

Rollator Article no*									Folded	Max user weight
	Min	Max	Seat height	Width	Seat width	Length	Wheel size	Weight (kg)	LxWxH (Min.)	Standard (kg)
Carl-Oskar										
11150	770	950	640	570	370	820	300x45	11,1	820x260x770	150
11250	770	950	640	620	420	820	300x45	11,3	820x260x770	150
11350	770	950	640	670	470	820	300x45	11,4	820x260x770	150
11450	770	950	640	720	520	820	300x45	11,5	820x260x770	150
Carl-Oskar air pumped wheels										
11151	770	950	640	570	370	820	300x45	11,1	820x260x770	150
11251	770	950	640	620	420	820	300x45	11,3	820x260x770	150
11351	770	950	640	670	470	820	300x45	11,4	820x260x770	150
11451	770	950	640	720	520	820	300x45	11,5	820x260x770	150
Carl-Oskar reinforced										
61225	770	950	640	620	420	820	300x45	13,2	820x260x770	275
61325	770	950	640	670	470	820	300x45	13,3	820x260x770	275
61425	770	950	640	720	520	820	300x45	13,4	820x260x770	275
Rebel										
22140	750	930	620	570	370	750	200x40	9,5	750x250x740	140
22240	750	930	620	620	420	750	200x40	9,6	750x250x740	140
22340	750	930	620	670	470	750	200x40	9,7	750x250x740	140
22440	750	930	620	720	520	750	200x40	9,9	750x250x740	140
Rebel low										
22141	680	860	540	570	370	750	200x40	9,2	750x250x680	140
22241	680	860	540	620	420	750	200x40	9,4	750x250x680	140
22341	680	860	540	670	470	750	200x40	9,5	750x250x680	140
22441	680	860	540	720	520	750	200x40	9,7	750x250x680	140
Rebel reinforced										
71221	750	930	620	620	420	750	200x40	11,5	750x250x740	250
71321	750	930	620	670	470	750	200x40	11,6	750x250x740	250
71421	750	930	620	720	520	750	200x40	11,8	750x250x740	250

(All measurements in mm)

* When ordering, make an addition after the part number for the package size, QX, where Q is for quality and X for the size of the package. Please see www.humancaregroup.com for more information.

Accessories

Accessories	Art. no.	
Handle Cobra H	00022	
Handle Cobra V	00021	
Basket with handles	10601	Important! Max load 10 kg
Tray for 57	10651	Max load 2 kg
Tray for 62, 67, 72	10602	Max load 2 kg
Cane holder	10603	
Back support for 57, 62, 67	10621	
Back support for 72	10655	
Oxygen holder 1-5 l (Not Rebel low)	10615	Max load 5 kg
Slow down brake	10620	
One-hand brake	10652	
Cerb climber (Not for reinforced models)	10612	
Splash guard (For Carl-Oskar)	10608	

See respective assembly instruction.

Table 1. Possible accessory combinations

	Basket	Tray	Cane holder	Back support	Oxygen holder	Slow down break	One-hand brake	Cerb climber	Splash guard
Basket	-	x	x	x	x	x	x	x	x
Tray	x	-	x	x	x	x	x	x	x
Cane holder	x	x	-	x	x	x	x	x	x
Back support	x	x	x	-	x	x	x	x	x
Oxygen holder	x	x	x	x	-	x	x	x	x
Slow down brake	x	x	x	x	x	-	x	x	-
One-hand brake	x	x	x	x	x	x	-	x	x
Cerb climber	x	x	x	x	x	x	x	-	-
Splash guard	x	x	x	x	x	-	x	-	-


- = Not applicable

User guide


- Read all warnings and instructions carefully before using the rollator.
- Ensure that the rollator is properly unfolded before use.
- The rollator may only be used as a support for walking or sitting.
- Never stand on the rollator.
- Do not use the product on stairs or escalators.
- Be extra careful when using the rollator on slopes or uneven surfaces.
- Never sit on the rollator and roll yourself forward with your feet.
- Distribute weight evenly to prevent the rollator from tipping when you sit on it.
- Always set the parking brake before using the rollator for support while sitting down or getting up.
- Do not place anything on a folded up rollator when it is in storage or being transported as this may cause damage to it.
- Do not store the rollator close to a heat source as this could cause damage to some details.
- Be careful when you handle the rollator if it has been exposed to abnormally high or low temperatures to avoid injuries.
- Do not store the rollator outdoors.
- Save the user instructions for future use.

Unfolding


1. Move handle outwards.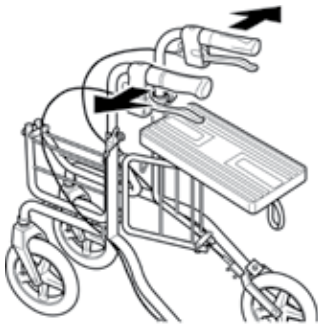

2. Push seat forward.

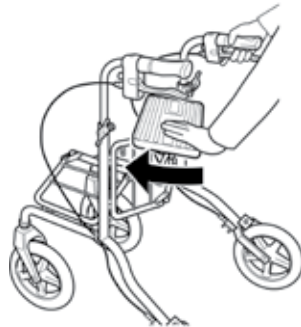
3. Important! Check that the lock grips the gate.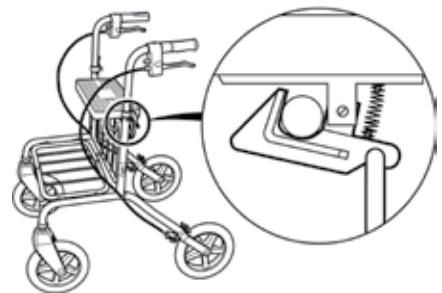


Folding

1. Pull the red loop until the lock frees the gate.

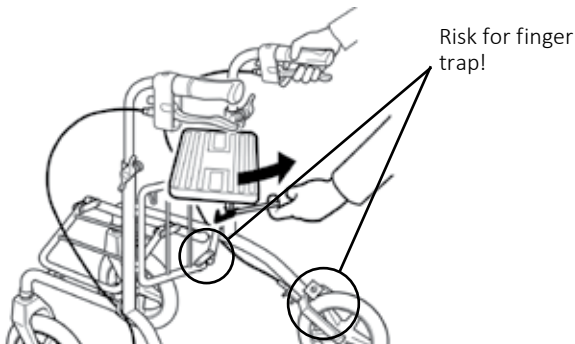
2. Pull seat backwards and move handle inwards.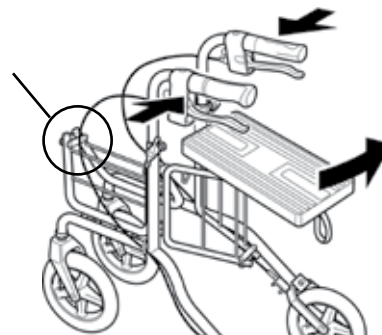


Adjustment of handles


Loosen wingnut and adjust handgrip to wrist height. Lock wingnut.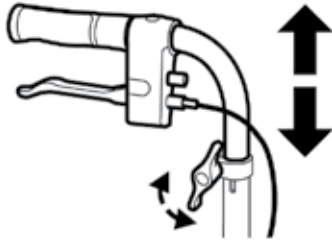

Important! The handgrips should not be heightened above the "0" mark on the sides of the handle pipes. The handles must be faced backwards. The mark must be aligned with the stroke on the clamp at the side of the pipes to avoid unstableness and dangerous situations.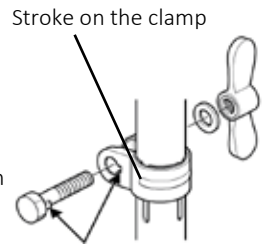

On the reinforced model, the strengthening bar should also be adjusted.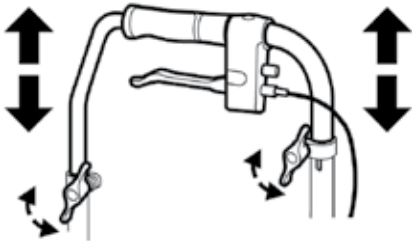


Warning! The handles must not be pointing outwards because of increased tip risk. The angle between the lengthwise line of the handles and the forward direction of the rollator must always be 180°.

Important! On assembly of the wing nut component, it is important that the base of the screw head engages fully the groove in the recess.

Parking brake

Pull brake lever downwards when parked.


Warning! Engage parking brake before sitting.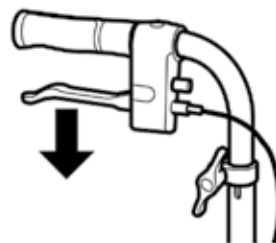

Brake

Pull brake lever upwards to brake.

Note! Continuous braking leads to excessive wear of wheels and brakes

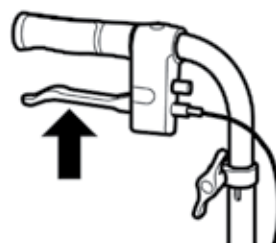Brake adjustment

Loosen locking nut; adjust brake by loosening/ tightening the adjustment casing. After adjustment tighten locking nut.

Cleaning and service

Important!

When changing parts of the rollator, use only Human Care parts to ensure continued CE validation.

Important!

Do not alter the rollator yourself.

Important!

Check the rollator regularly to ensure its continued safety to the user.

Regular checks should be carried out as recommended by the instructions detailed in:

Carl-Oskar, art. no.: 99211

Rebel, art. no.: 99212.

For more information on servicing, reconditioning, spare parts and accessories, see Human Care's website: www.humancaregroup.com or contact us at the address, phonenumber or e-mail address according to back page.

Warranty Terms

The Human Care HC AB warranty covers defects in material and manufacture of the walking aid. Wear on wheels, brake parts and handles are not included. Finish damages which have occurred after delivery are not covered. The warranty presumes that the product has been used in the intended way and that it has been regularly reconditioned in accordance with our reconditioning instructions. The ID number of the product must also be identifiable. Furthermore, the product must not have been modified or customized by any party other than Human Care HC AB. The warranty period is one (1) year or according to agreement.

For complaints, please contact us for instructions on how to proceed.

For complaints, the following information should be indicated: product ID number; the date of the last reconditioning; the name of the consignor contact; a description of the fault.

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